

LUMOHS QUICK START: BATTERY INSTALLATION AND REPLACEMENT

Unscrew (twist counter clockwise) the rear end cap opposite the blade side of lumohs™ scalpel handle. Remove the rear end cap. The barrel of the lumohs is marked with the battery orientation and shows where the battery packs are located. Remove batteries and dispose according to facility protocol. Insert new battery packs into barrel following the orientation symbol on the barrel. Ensure spring on circular metal end plate is positioned in the back of the rear end cap with the spring in the center of the end cap. Screw rear end cap back into place by twisting end cap clockwise. The last quarter turn clockwise will cause lumohs to turn on. Turn lumohs device on and off to ensure that battery replacement was done correctly.

ATTENTION:

USE ONLY AUTHORIZED LUMOHS BATTERY PACKS.

DO NOT USE NICKEL HYDRIDE RECHARGEABLE BATTERIES. DO NOT USE LITHIUM BATTERIES. USE ONLY THE LUMOHS BATTERY PACKS.

DO NOT INSTALL BATTERIES INTO LUMOHS HANDLE UNTIL HANDLE FULLY COOLED AFTER MOIST HEAT STERILIZATION.

DO NOT INSTALL BATTERIES BACKWARDS- SEE DIRECTIONAL DIAGRAM ON HANDLE.

WARNING:

EXPLOSION HAZARD – REMOVE BATTERIES BEFORE STERILIZATION. DO NOT AUTOCLAVE WITH BATTERIES IN HANDLE OF LUMOHS DEVICE.

EXPLOSION HAZARD - DO NOT CHARGE BATTERIES. DO NOT PUT BATTERIES IN A FIRE OR MIX WITH OTHER BATTERY TYPES AS THIS MAY CAUSE EXPLOSION OR LEAK CAUSING INJURY. DO NOT OPEN OR MANIPULATE LUMOHS BATTERY PACKS.